

Figure 1

The *AudioLink® MI-2* has an ON/OFF switch on the side of the case. Move this switch toward the ON position to activate your loop receiver (Fig. 1). A red indicator lamp on the front of the loop receiver tells you that it is active and working.

**Power (on/off):**

**For additional service information contact:**

**ELT Design**

Unit 103 - 20540 Duncan Way  
 Langley, BC  
 V3A 7A3 Canada  
 604/514-0053; 866/845-2537; fax: 604/514-0037

**ELT Group**

1820 West 10th Street, unit 2  
 Riviera Beach, FL, 33404 USA  
 561/655-8523; 800/286-3481; fax: 561/832-7273


Try to keep the *AudioLink® MI-2* in a vertical position in order to provide the clearest possible sound within the inductive loop system.

The *AudioLink® MI-2* is a pendant style receiver that can be worn around the neck with the lanyard, or in a pocket using the clip. It is designed to be used with earphones, earbuds, or headphones that have a 3.5mm mono plug.

**Using the *AudioLink® MI-2***

Thank you for purchasing an *AudioLink®* MI receiver. We are confident you will find this product greatly enhances your listening pleasure of movies, plays, worship services and in many other locations where you will find compatible induction loop systems installed.


***AudioLink® MI-2***  
 Magnetic Induction Loop Receiver

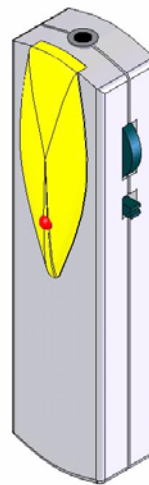

**USER GUIDE**

## VOLUME ADJUSTMENT


Volume (gain level) is adjusted by moving the thumbwheel located on the side of the case. To increase or decrease volume, adjust as shown in (Fig.2)

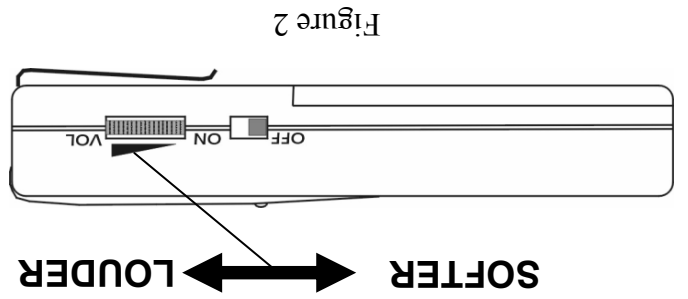

Page 4

## INSTALLING BATTERIES

The *AudioLink® MI-2* loop receiver uses two "AAA" or UM-4 type batteries. Alkaline batteries are recommended for best performance. To install or change the batteries, just slide the battery cover (located on back of receiver) downwards to remove. Install the batteries with polarity as shown in (Fig. 3). Replace the battery cover on the receiver.

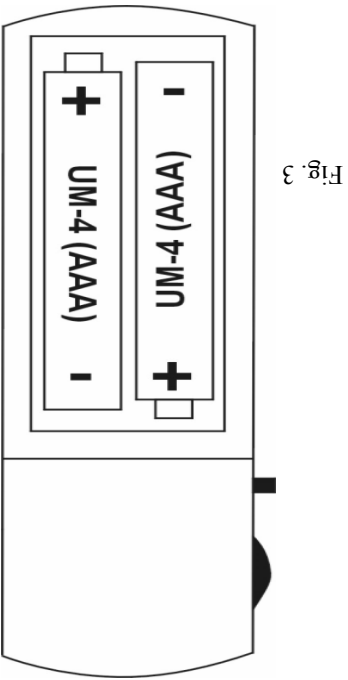

Page 5

## SPECIFICATIONS

### *AudioLink® MI-2* Receiver

<b>Weight:</b>	1.7 oz (47 g) with batteries
<b>Material and Color:</b>	ABS plastic, Grey and Yellow
<b>Battery Type:</b>	two UM-4 (AAA) Batteries
<b>Operating time:</b>	35+ hours
<b>Audio Frequency Response:</b>	150—6000 Hz +/- 3 dB
<b>Signal to Noise Ratio:</b>	45 dB in 110 mA/m magnetic field
<b>Controls:</b>	Volume thumbwheel; ON/OFF
<b>Output termination:</b>	3.5mm mono audio jack (TS)
<b>Output Level:</b>	250 mVrms (32 ohm load, magnetic field strength 100 mA/m)
<b>Compatibility:</b>	Compatible with installed room loops, countertop loops, and portable loop systems.
<b>Power Switch:</b>	Slide action switch
<b>Power Indicator:</b>	Red LED on front of housing
<b>Warranty:</b>	2 years parts & labor* *90 days on headphone and accessories

Page 6

## WARRANTY AND REPAIR

Your new *AudioLink® MI-2* pendant loop receiver is a high quality device. The warranty covers both parts and labor for a period of 2 years from date of retail purchase for the headset and charger, 6 months for battery packs, and adapter related failures. The warranty period is established by the date on your purchase receipt. During the warranty period, the manufacturer will (at its sole discretion) repair or replace the charger or headset should it fail due to a defect in materials or workmanship.

Your obligation under the warranty is to contact your authorized ELT Group service center for a "Return Authorization Code" and then return the unit to the service center with a copy of your purchase receipt. ELT Group will pay the return shipping to the service center. ELT Group will pay the return shipping. Be sure to carefully pack the unit(s) to avoid damage in transit and use a traceable transportation method such as UPS, FEDEX or insured mail.

Beyond the warranty period, service will be provided for a nominal fee.

Page 7